

NEW ATLAS FORCEPS

Innovation in the palm of your hand

P. 18

N° 01

HIGHLIGHTS

P. 1 | SEE YOUR PATIENTS -
NOT YOUR
INSTRUMENTS

P. 7 | AN INFECTION
PREVENTION SOLUTION
BUILT AROUND YOU

P. 11 | THE NEW FRONTIER OF
RESTORATIVE
DENTISTRY

P. 28 | LET'S BE CLEAR:
ENHANCING ALIGNER
TREATMENTS

Visit us on
hu-friedy.eu

HUNGARY

All prices are VAT included. Valid from February 14th until April 30th, 2020

Hu-Friedy

How the best perform

INDEX

SEE YOUR PATIENTS -
NOT YOUR INSTRUMENTS

AN INFECTION PREVENTION
SOLUTION BUILT AROUND YOU

THE NEW FRONTIER OF
RESTORATIVE DENTISTRY

INNOVATION IN THE PALM
OF YOUR HAND

SWISS PERIO KIT

LET'S BE CLEAR:
ENHANCING ALIGNER
TREATMENTS



New HD Black Line Mirrors
with Tami Wanless and Carlo Poggio
P. 1



Why re-evaluate your Instrument Management?
P. 7



Adhesthetics Theca Kit
with Federico Ferraris
P. 11



New Atlas Forceps
with Francesco Merlettini
P. 18



with Rino Burkhardt
P. 24



Clear Collection Instruments
with S. Jay Bowman and Tom Freston
P. 28

HU-FRIEDY
PROMOTIONS



SEE YOUR PATIENTS - NOT YOUR INSTRUMENTS


New HD Black Line Mirrors

Glare interferes with visibility. Reducing this disruption to your visibility allows you to see more detail and offer more accurate patient care. HD Black Line Mirrors are engineered to optimize clinical outcomes by delivering superior visibility throughout any dental procedure. The reduced glare properties of the Diamond Like Carbon (DLC) coating and clarity of the HD mirror glass facilitates quicker and more accurate visibility intraorally.

“ I wear loupes with a LED light and noticed a significant difference in the amount of glare reflecting back into my field of vision during patient care, allowing me to see more detail. I realized, at the end of my clinical day, I had less eye strain when I used the HD Black Line Mirrors. ”



Tami Wanless,
RDH, MED

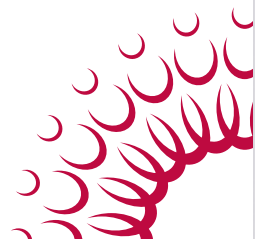

SEE YOUR PATIENTS - NOT YOUR INSTRUMENTS

New HD Black Line Mirrors

Diagnostic is the very starting point of every dental procedure and as a famous proverb states “well begun is half done”. Having a clear and precise diagnosis is needed in order to correctly plan the necessary treatment.

Sight is the first sense that every clinician uses, therefore having the best possible vision is crucial.

With this in mind, Hu-Friedy leveraged the success of HD Mirrors, Blackline and XTS product lines, to create **HD Black Line Mirrors**.

Hu-Friedy's newest innovation was engineered to optimize clinical outcomes by delivering superior visibility throughout any dental procedure.

Designed for enhanced performance, Hu-Friedy's HD Black Line Mirrors have a **Diamond Like Carbon (DLC) coating**, which **reduces glare up to 80%**** compared to a standard metal mirror heads and handles. This helps to reduce strain and fatigue, creating a more ergonomic mirror, as the user does not need to adapt their viewing position due to unwanted shine produced by traditional metal mirror handles or frames.

Additionally, the **black matte finish provides enhanced contrast and visual acuity** within the oral cavity. This creates a distinct contrast between the instrument, the tooth and/or the surrounding tissue allowing for easy identification intraorally.

The DLC coating in combination with the **superior brilliance and color of Hu-Friedy's proprietary HD Mirror glass** facilitates quicker and more accurate visibility of the mouth.

The benefits of Hu-Friedy's HD Mirror glass:

- **113 %* reflection factor** for exceptional image clarity
- **50 %* brighter** than other front surface mirror glass
- **38.5 %* brighter** than rhodium coated mirror glass
- **Front surface reflection** provides a sharp, distortion-free image*

The large diameter handle offers increased control creating an ergonomically friendly option to provide the clinician maximum comfort and reduces hand fatigue. ■

** Data on file and available upon request.*



| Use of HD Black Mirror to facilitate the digital impression.

| Quote and clinical image is courtesy of Dr. Carlo Poggio.



Dr. Carlo Poggio

Tips & Tricks in Digital:

“A very simple “digital dentistry” issue: when dealing with intraoral scans in the posterior areas, it is often necessary to retract the tongue. Unfortunately our usual retractor, the dental mirror, has a metal reflective back surface, which is not ideal for light reflection; scanning may be interrupted or get distorted.

By using Hu-Friedy's new HD Black Line Mirror I definitely solved the light reflection issues.”

REDUCES GLARE UP TO 80%**



***When comparing the Hu-Friedy DLC coated mirror head and handle to the Hu-Friedy non-coated stainless steel mirror head and handle.*

HU-FRIEDY
PROMOTIONS



SEE YOUR PATIENTS - NOT YOUR INSTRUMENTS

New HD Black Line Mirrors


HU-FRIEDY'S NEW HD BLACK LINE MIRRORS ARE NOT JUST FOR HYGIENE EXAM PROCEDURES. COMPLETE YOUR KIT NO MATTER THE SETUP!



General Exam



Basic Hygiene

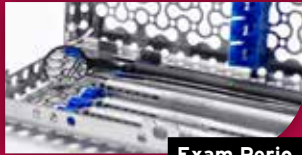

Exam Perio



Restorative



Endodontic



Surgical



CONE SOCKET THREAD



10.302,24 FT

MIR5DSHDB
#5 Double-sided HD Mirror, Cone Socket, Black Line, Single Pack



8.370,57 FT

MIR5HDB
#5 Single-Sided HD Mirror, Cone Socket, Black Line, Single Pack



10.302,24 FT

MIR5DSHDBE
#5 Double-sided HD Mirror, European, Black Line, Single Pack



8.370,57 FT

MIR5HDBE
#5 Single-Sided HD Mirror, European, Black Line, Single Pack

MH6B

Cone Socket Mirror Handle, Satin Steel, Black Line

11.160,76 FT



MHE6B

European Mirror Handle, Satin Steel, Black Line

11.170,66 FT

LAUNCH PRICE

Double sided mirror + Handle 17.814,29 FT (retail 21.463 FT)

Single sided mirror + Handle 16.140,18 FT (retail 19.531,33 FT)

BE SAFE AND IMPROVE YOUR WORKFLOW!



DESCRIPTION

PARTCODE

IMS Infinity Series 1/4 DIN Cassette 5 Instruments, Blue

IME14DIN58

#6 European Simple Stem Mirror Handle BLACK LINE

MHE6B

#5 Single-Sided European HD Mirror BLACK LINE

MIR5HDBE

#23 Shepherd's Hook Explorer SE, handle #6

EXS236

#CP-15 UNC Color-Coded Probe Metal SE, handle #6

PCPUNC156

#2 College Dressing Pliers

DP2

DIAGNO KIT

85.809,07 FT


HU-FRIEDY PROMOTIONS





DISCOVER HOW OUR IMS SYSTEM CAN IMPROVE YOUR SAFETY AND ORGANIZATION IN THE NEXT ARTICLE

BE SAFE AND IMPROVE YOUR WORKFLOW!

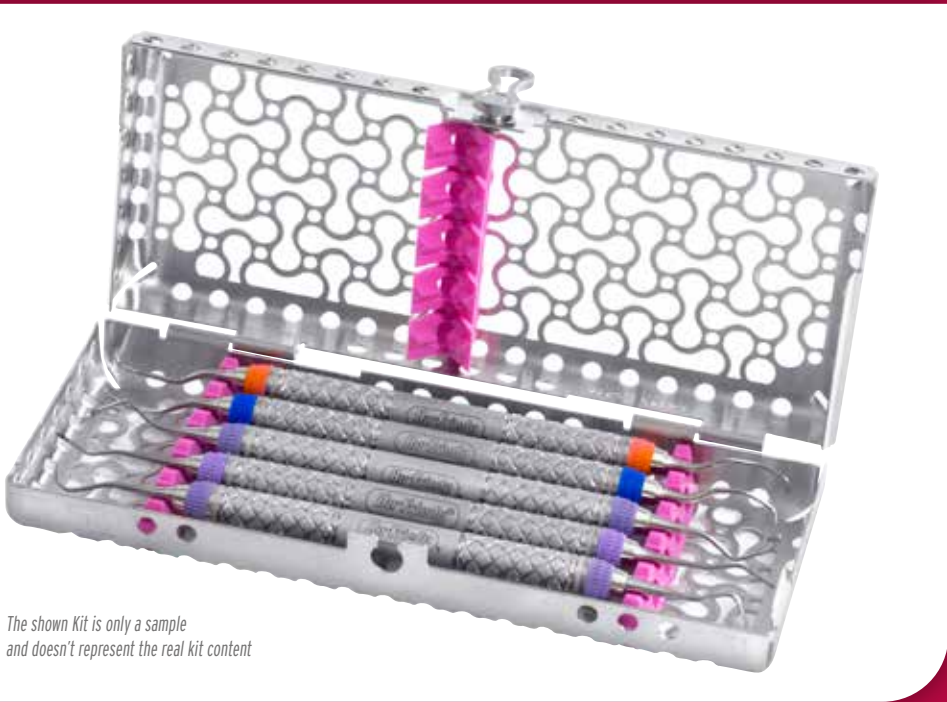

The shown Kit is only a sample and doesn't represent the real kit content

DESCRIPTION	PARTCODE
IMS Infinity Cassette 1/4 DIN, 5 Instruments, blue	IME14DIN58
Scaler #23 EE 2.0, handle #9	SM239E2
Curette Gracey #5/6 EE 2.0 MF oral/labial, yellow, handle #9	SAS5/695E2
Curette Gracey #7/8 EE 2.0 MF oral/buccal, green, handle #9	SAS7/897E2
Curette Gracey #13/14 EE 2.0 AF distal, blue, handle #9	SRP13/1498E2
Curette Gracey #15/16 EE 2.0 AF mesial, orange, handle #9	SRP15/1693E2

PERIO KIT

137.320,27 FT



AN INFECTION PREVENTION SOLUTION BUILT AROUND YOU

Why reevaluate the way you manage your instruments?

It may seem like a small step, but an Instrument Management System can have a big impact on your practice.

Keeping instruments organized and managing them properly improves the flow of your dental clinic.

The sterilization area can run more efficiently, and staff members remain focused on the patient while providing the best level of care.



One of the most heated topics in the dental field, and very often discussed among clinicians, is infection prevention and the best protocols to follow during instrument reprocessing within the dental clinic.

A primary infection control goal for health care is to provide sterile instruments during patient treatment in order to ensure safety. Not only are the patients exposed to the risk of blood borne pathogens, but also clinicians and dental assistants themselves. According to the CDC, (Centers for Disease Control and Prevention, USA) and European Recommendations, dental practitioners, hygienists and

assistants need annual trainings about these serious issues as they never stop benefitting from helpful reminders.

Moreover, **because of the increasing problem of bacterial resistance, infection control protocols are constantly under the spotlight** and more and more they are becoming the utmost priority in the dental field. While proper instrument sterilization remains the end goal of infection control, considerations can be done concerning the technologies and protocols that minimize the risk of cross-contamination during the various steps of instrument reprocessing. ►

AN INFECTION PREVENTION SOLUTION BUILT AROUND YOU



A better practice with the Instrument Management System (IMS)

Technological advancements in infection control equipment have led to an ongoing evolution of tools and the Instrument Management System (IMS) by Hu-Friedy. IMS offers a high standard of organization in the dental office while combining the cleaning, sterilization and storage of dental instruments.

Using Hu-Friedy cassettes organized by procedure offers an alternative approach

for the whole dental office, meaning a sensible improvement in the team workflow.

Keeping organized instruments, accessories and disposable in a single integrated system per procedure, allows clinicians not only to be efficient and increase their productivity, but also to prevent many of the frequent sharp exposures encountered during hand scrubbing, drying, sorting and packaging of instruments.

The number of advantages of using IMS solution are tangible. For example, when the instruments are locked into a cassette,

their tips are safe and are less susceptible to damage, breakage or misplacing and the risk of an injury during transportation from the patient operatory to the sterilization area is minimal. Using cassettes enhances chairside efficiency and the productivity of the entire dental clinic. Instruments are organized into procedural set-ups with a color-coded silicone rail system that makes visual identification easy. This saves minutes with every patient, allowing additional time for the doctors that are caring for them.

Patients want and deserve optimal infection control, and a professional chairside image through the presentation of sterile packages, they are aware that the instruments are clean, sterile and ready to be used for their procedures according to proven safety protocols and regulations.



Streamline reprocessing by eliminating the time-consuming steps that slow down staff

Depending on the clinic and the practitioners' needs, Hu-Friedy's Instrument Management System (IMS) can be adapted and customized while avoiding forgetting every step of the cleaning procedures and promoting the proper flow

of dirty to clean. Clinicians who have implemented the IMS solution in their dental practice, benefit every day from the advantages of a simplified system that has been customized according to their clinical expectations, to how many patients they visit per day and how many chairside they have in the clinic. Clinicians have noticed that investing in IMS is necessary during implementation, considering dental offices may need to upgrade their sterilization area. However, the increased efficiency and productivity of saving time for every single procedure pays off the investment in the long run. ►

HU-FRIEDY PROMOTIONS



BUILD YOUR INSTRUMENT MANAGEMENT SYSTEM WITH US!

Choose your preferred solution:

• IMS INFINITY CASSETTES AND WRAPS

CASSETTES:

IME14DINS*
IMS Infinity 1/4 DIN
Cassette for 5 Instruments



IME12DIN1*
IMS Infinity 1/2 DIN
Cassette for 10 Instruments

IMEDIN13*
IMS Infinity 1 DIN
Cassette for 13
Instruments with
accessory area



WRAPS:

IMS-121IMH
Autoclave Wrap for 1/4 DIN
Cassettes (100 sheets)

IMS-1212MH
Autoclave Wrap for 1/2 DIN
Cassettes (100 sheets)

IMS-1210MH
Autoclave Wrap for 1 DIN and
Container Cassettes (100 sheets)



TAPE:

IMS-126*
IMS Autoclave Monitor Tape, 55 mm



* You can choose between different colors and procedure texts. Check it downloading IMS Catalogue, page 28.



ADD YOUR TUBS:

IMS-140*
IMS Signature Series Complete Tub



27.858,97 FT

ORGANIZE YOUR PRACTICE BY COLOR CODING

*CHOOSE THE COLOR.
Just add the color coding number to the last part of the partcode (replace "-" with the number)

0 = Red
1 = Grey
2 = White
3 = Orange
4 = Violet
5 = Yellow
6 = Lavender
7 = Black
8 = Blue
9 = Green
11 = Ocean Blue

ORGANIZE YOUR PRACTICE BY COLOR CODING

*CHOOSE THE COLOR.
Just add the color coding number to the last part of the partcode (replace "-" with the number)

0 = Red
1 = Grey
2 = White
3 = Orange
4 = Violet
5 = Yellow
6 = Lavender
7 = Black
8 = Blue
9 = Green
11 = Ocean Blue

CASSETTES:



IMECC12_S*
IMS Surgical Container Cassette for 12 Instruments

Cassette + Container M + Filter + Indicators Label + Seal
212.591,02 FT

CONTAINERS:



IMDINCOOM
IMS DIN Container
312 x 190 x 050 mm, M version



IMDINC03M
IMS DIN Container, For Max. 3 Container
Cassettes, 312 x 190 x 135 mm

Add 8.617 FT
for Container for 3 Cassettes

ACCESSORIES:

IMCO-FILIM
IMS Container Accessories
Filter 95 x 215mm (PTFE)
2pcs./pack., M version

IMCO-IND
IMS Container
Accessories Indication
Label - 100pcs./pack.

IMCO-CO
IMS Container Identification
Label, blue (Laser marking
available at additional charge)

IMCOM-SEAL
IMS Container Accessories
Seal 100pcs./pack., blue,
M version

ADD YOUR TUBS:

IMS-140*
IMS Signature
Series
Complete Tub



27.858,97 FT

AN INFECTION PREVENTION SOLUTION BUILT AROUND YOU

A complete turn-key solution for a long-lasting relationship

During the implementation process, Hu-Friedy representatives offer their expertise to help clinicians build their own customized solution. First, the equipment needed is determined, then the number of cassettes that are necessary for each procedure. The implementation takes place one step at a time with the constant support and follow-up of Hu-Friedy.

It's an ongoing process that is possible only by working together and even once the process is completed, Hu-Friedy continues to organize educational trainings about IMS and Infection Control and develops new up-to-date tools that can be added to the implementation at any time. Practitioners still benefit from the IMS system even after 20 years through increased organization between one procedure and another, a more efficient workflow saving considerable time saving and improved safety and infection control. ■



DISCOVER
MORE

Read what Experts says about
INSTRUMENT MANAGEMENT SYSTEM

imsuser.hu-friedy.eu/en/resources/ims-best-practice-and-testimonials/



ORGANIZE YOUR CLAMPS



IMS-1273
IMS Accessory Box, large

20.089,37 €

IMS-1271
IMS Accessory Box, small

14.873,86 €

THE NEW FRONTIER OF RESTORATIVE DENTISTRY

Federico Ferraris Adhesthetics Theca Kit

The **Adhesthetics Theca kit** is comprised of four unique instruments designed by Dr. Federico Ferraris in collaboration with Hu-Friedy. It is a unique set of instruments created for use during the clinical phases of both direct and indirect restorative dentistry to help clinicians achieve positive results through clinical excellence.



The Main Shaper, the Fine Shaper, the 3D Shaper and the Direct Caliper are all made using XTS

technology, a dark **Aluminum Titanium Nitride (AlTiN)** coating that offers superb contrast in comparison to composite materials and tooth structure.

The coating is also incredibly slick, resisting the adhesion of sticky composite materials. The smooth handle is not only ergonomic and lightweight, but each instrument has its own distinct set of colour cones, making instrument identification simple. The set of four instruments includes an IMS Cassette, which prevents the instruments from being damaged during transportation and sterilization protocols. ►

“ *Incorporating these techniques into everyday procedures allows for predicable results in direct and indirect restorative dentistry. Constantly challenging yourself and continuously making personal improvements is the personal motto of Federico Ferraris:* **Do better than yesterday.** ”



Born on 10th of August 1974 in Alessandria, Italy.

Graduated in Dental School at Genoa University as DDS in 1999. ADHESTHETICS founder.

International speaker at dental congresses and courses in more than forty Countries.

Active Member of the following Academies: EAED (European Academy of Esthetic Dentistry), AIC (Italian Academy of Conservative), IAED (Italian Academy of Esthetic Dentistry) and SCAD (Society for Color and Appearance in Dentistry).

Member of AARD (American Academy of Restorative Dentistry) and IAAD (International Academy for Adhesive Dentistry).

He is President Elect of SCAD for the biennium 2021-2022 and of AIC for the biennium 2022-2023 DSD (Digital Smile Design) Master since 2013.

Member of the Editorial Board of IJED (International Journal of Esthetic Dentistry) since 2010.

Co-author of several Italian and International scientific publications.

Dental practice in Alessandria, focusing in Operative Dentistry and Prosthodontics.

www.federicoferraris.com



THE NEW FRONTIER OF RESTORATIVE DENTISTRY

Federico Ferraris Adhesthetics Theca Kit

HU-FRIEDY
PROMOTIONS



TNFF1/2 Main Shaper

FF1

Manipulation of composite on buccal surfaces during direct anterior restorations, Class IV and V cases*, direct veneers restorations and general esthetic restorations.

The tip's lance shape and flexibility allow for it to adapt to restorative material (Fig. 6) and is also effective in removing excess material from the vestibular region by using its edge (Fig. 7).



FIG. 6



FIG. 7

Its flexibility allows for restorative materials to be manipulated with a delicate touch. It is sturdier than a standard brush however less rigid than a standard spatula.

FF2

Application of medium-great quantity of material directly on the tooth or as a material carrier. Its round shape and rigidity allow for the manipulation of restorative materials in wide cavities. It is also effective when placing material on wide surfaces (Fig. 8).

Dimensions of working part: length 11,5 mm; width 2,5 mm and thickness 0,4mm.



FIG. 8

TNFF3/4 Fine Shaper / Cord Packer

FF3 & FF4

Modelling of restorative materials on anterior and posterior teeth as well as the placement of retraction cord. A distinctive characteristic of this spatula (FF3 straight and FF4 angled) is its flexibility. Its thin design allows for precise modelling (Fig. 9, 10, 11, 12).



FIG. 9



FIG. 10



FIG. 11

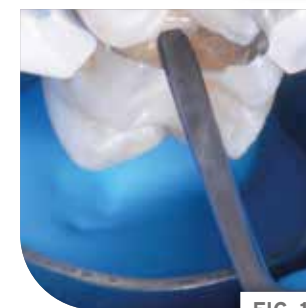


FIG. 12

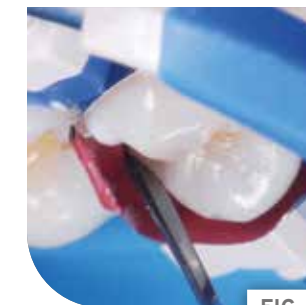


FIG. 13



FIG. 14

It is particularly useful for deep cavities. It is also helpful during procedures where high magnification is used because its length allows for easier access to these deep regions while not interfering with the clinician's field of vision (Fig. 13). Retraction cord can be placed precisely with the thin instrument tip (Fig. 14). ►

* According to Black's classification of cavities. Black's classification of cavities. Oxford Reference. Retrieved 6 Nov. 2019, from <https://www.oxfordreference.com/view/10.1093/oi/authority.20110803095509657>.

TNFF1/2

TNFF3/4

THE NEW FRONTIER OF RESTORATIVE DENTISTRY

Federico Ferraris Adheshthetics Theca Kit

HU-FRIEDY
PROMOTIONS



TNFF5/6 3D Shaper

FF5

Anatomical modelling of restorations of posterior cusps (Fig. 15, 16, 17) or of anatomical dentinal or superficial elements on anterior teeth (Fig. 18). The unique teardrop shape of this point makes it excellent for sculpting materials in both posterior and anterior cases.



FIG. 15



FIG. 17



FIG. 16



FIG. 18

FF6

Sculpting thin sulcus as well as finishing of the margins and thin modellings on posterior and anterior direct restorations (Fig. 19, 20, 21, 22). Application of fluid material in deep cavities (Fig. 23), positioning of flowable materials in occlusal sulcus (Fig. 24), coating low viscosity materials (Fig. 25), removal of excesses of high viscosity cementation material (Fig. 26) and control of margins after cementation (Fig. 27). point makes it excellent for sculpting materials in both posterior and anterior cases.



FIG. 20



FIG. 23



FIG. 26



FIG. 21



FIG. 24



FIG. 27



FIG. 19

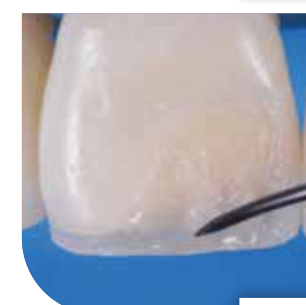


FIG. 22



FIG. 25

Its distinctive curved shape allows the tip to easily reach areas in both the anterior and posterior arch. Fine details can be created in restorative materials due to the tip's thin design. ►

TNFF5/6

THE NEW FRONTIER OF RESTORATIVE DENTISTRY

Federico Ferraris Adhesthetics Theca Kit

HU-FRIEDY
PROMOTIONS



TNFF7/8

Direct Caliper

FF7 e FF8

Assess thickness of enamel margin in direct restorations on anterior teeth (Fig. 28), assess thicknesses of direct and indirect restorations during various clinical phases (Fig. 29), creating definition of dentinal curving (Fig 30).



FIG. 28

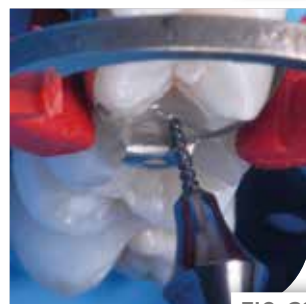


FIG. 29



FIG. 30

This instrument is a restorative probe with a flat head (not rounded, as most periodontal probes) with grooves at 0.5mm increments. The tip is angled at 45° which when aligned against the preparation bevel, allows for the thickness of dentinal material and residual buccal enamel to be determined.

The FF7 can be used to assess an enamel thickness of 0.5mm (the universal thickness in the third medium of the crown), while the FF8 can be used to assess a thickness of 0.9mm (which is preferred in incisal areas, or times when the thickness of the natural enamel is of critical importance). The shape of the dentin in the incisal direction in addition to the thickness to the margin can be determined with this instrument. ■

ADHETHETICS THECA KIT TNFFDIN

128.348,74 FT



HD Black line Mirror sold separately

ADD 16.140,18 FT TO HAVE THE NEW HD BLACK MIRROR (MIR5HDBE + MHE6B)

INNOVATION IN THE PALM OF YOUR HAND

New Atlas Forceps

HU-FRIEDY
PROMOTIONS



Innovation at Hu-Friedy never stops

Before the official October 1st launch date of the new Atlas forceps, the European dental community got an early look at this new product line during the EAO Congress in Lisbon at the end of September.

Atlas forceps are another great example of how to meet a clinicians' needs during extraction procedures. They also represent a significant addition to a now **more complete and attractive line of extraction forceps**.

Our expanded portfolio provides the clinicians with a **wider range of possibilities** to better fit their needs when approaching extractions.

Within this portfolio, the Atlas forceps product line represents the highest value for your dollar, with its internal spring which maintains the forcep in an open position helping reduce manipulation during surgeries, enhancing comfort and control.



European
polished



European
matte



Presidential



Atraumair



Atlas


NEW





INNOVATION IN THE PALM OF YOUR HAND

New Atlas Forceps

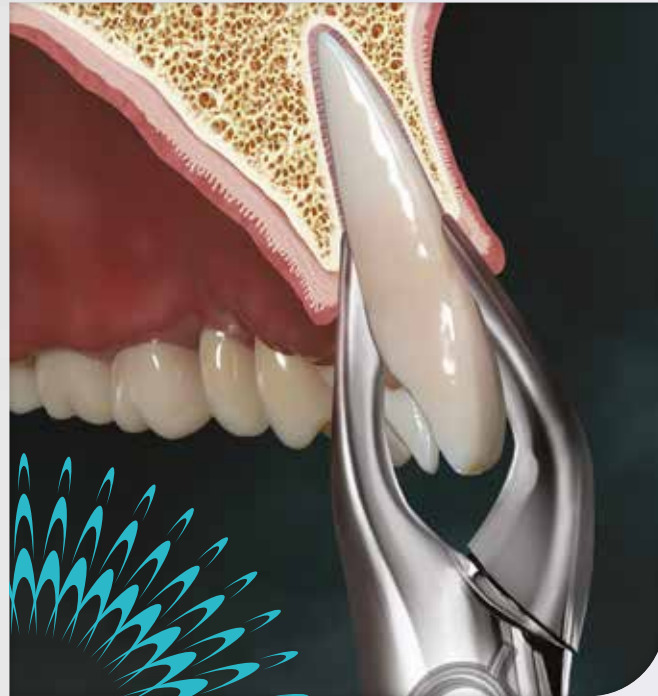

Be amazed by the enhanced comfort provided by Atlas forceps

It takes years of hard work, education and training to determine the proper balance of the firm yet delicate technique needed during an extraction procedure to ensure patient comfort.

Engineered to provide simplicity and precision, Atlas Forceps have the power to effectively perform even the most difficult extractions.

The innovative internal spring helps reduce hand manipulation, increasing comfort and control.

The Forceps offer a full arch solution through the 14 designs which are available in both standard and apical styles. ■

“ There are several advantages about using an Atlas extraction forcep, **but the feeling coming from handling it is definitely unique!** ”

F. Merlettini
EMEA Surgical
Product Manager



Main features and benefits:

- Internal springs maintain the forceps in the open position, **helping reduce manipulation during surgery, enhancing comfort and control**
- Apical designs provide a **firm, secure grasp of the crown and root**
- Thin, sharp tapered beaks for **ease in penetrating the PDL and apical progression***
- Innovative spring hinge ensures forceps are in the **open position during reprocessing**
- Laser marked hinge for **easier identification when bagged during cleaning and sterilization**
- Meticulously handcrafted from the **finest grade surgical stainless steel**

* Apical designs with tapered beaks during subgingival apical progression in an extraction

DID YOU KNOW?

All forceps should be kept lubricated. Regular use of proper lubricants, like Hu-Friedy Instrument Lubricant Spray (ILS), will prevent rust, corrosion and stiff joints to ensure smooth operation. **All hinged instruments should be sterilised in the open position.**

ILS

IMS Lubricant Spray for instruments
8oz/237ml Pump



INNOVATION IN THE PALM OF YOUR HAND

New Atlas Forceps

HU-FRIEDY
PROMOTIONS



F150IS

150
Upper
Universal



F151IS

151
Lower
Universal



F1IS

1
Upper
Anterior



F23IS

23
Cowhorn,
Lower
Molars



F53LIS

53L
Upper Molar,
Left



F53RIS

53R
Upper Molar,
Right



FAF1IS

1
Upper
Anterior,
Apical



FAF150IS

150
Upper
Universal,
Apical



F222IS

222
Lower
Molars



F69IS

69
Tomes
Upper/Lower
Roots



F88LIS

88L
Nevius
Upper Molar,
Left



F88RIS

88R
Nevius
Upper Molar,
Right



FAF151IS

151
Lower
Universal,
Apical



FAF222IS

222
Lower Molars,
Apical



LAUNCH PRICE
81.988,66 FT

(retail 96.583,50 FT)

SWISS PERIO KIT

by Dr. Rino Burkhardt

Main features and benefits:

- INDIVIDUALLY DESIGNED INSTRUMENTS

Refined working ends helps reduce the likelihood of tissue trauma

Ultimate manufacturing tolerances for precise handling

Easy to clean

- BLACK INSTRUMENTS / BLACK LINE EXTENSION

More contrast between the instrument and the tissue

Avoids light reflection

Optimal edge retention and enhanced lubricity

- BALANCED WEIGHT

More control and precision during delicate microsurgery procedures

HU-FRIEDY
PROMOTIONS



SwissPerio Education was founded by a group of periodontists and stands for a new training platform offering tailored courses on various topics in the domain of periodontology and related fields.

Events organized by SwissPerio are different from those of other providers as the transfer of disciplinary knowledge on an expert and evidence-based level is just one of the facets of the courses. The restricted number of participants provides room for personal interaction and open discussions, stimulating the trainees for critical thinking and teaching them in clinical decision-making.

Most of the SwissPerio events are related to clinical practice and, therefore, the third focus of SwissPerio Education is directed at the improvement of psychomotor manual skills

Dr. Rino Burkhardt



Rino Burkhardt, one of the Board Members of SwissPerio Education, is responsible for the training programs. He is an EFP (European Federation of Periodontology) certified specialist in periodontology and received his Masters degree from the Medical Faculty of the University of Berne, Switzerland. Since 1996 he runs a private clinic in Zurich, limited to periodontology and implantology. Additionally, he holds an Honorary Professorship at the University of Hong Kong and a lectureship at the University of Zurich. He is an active member of the Swiss Society of Periodontology (SSP), the European Academy of Aesthetic Dentistry (EAED), the European Association for Osseointegration (EAO), and Board Member of the Swiss Society of Oral Implantology (SGI). He published numerous book chapters and articles and won a research prize from the EFP.

swissperio.com

swiss+perio



SWISS PERIO KIT

by Dr. Rino Burkhardt




SPTPDSSPV+SPSPV+SPNHDPVN =

ESSENTIAL SWISS PERIO KIT

BUY SPTPDSSPV+SPSPV+SPNHDPVN | **402.560,03 FT** (retail 729.141,04 FT)

HU-FRIEDY
PROMOTIONS

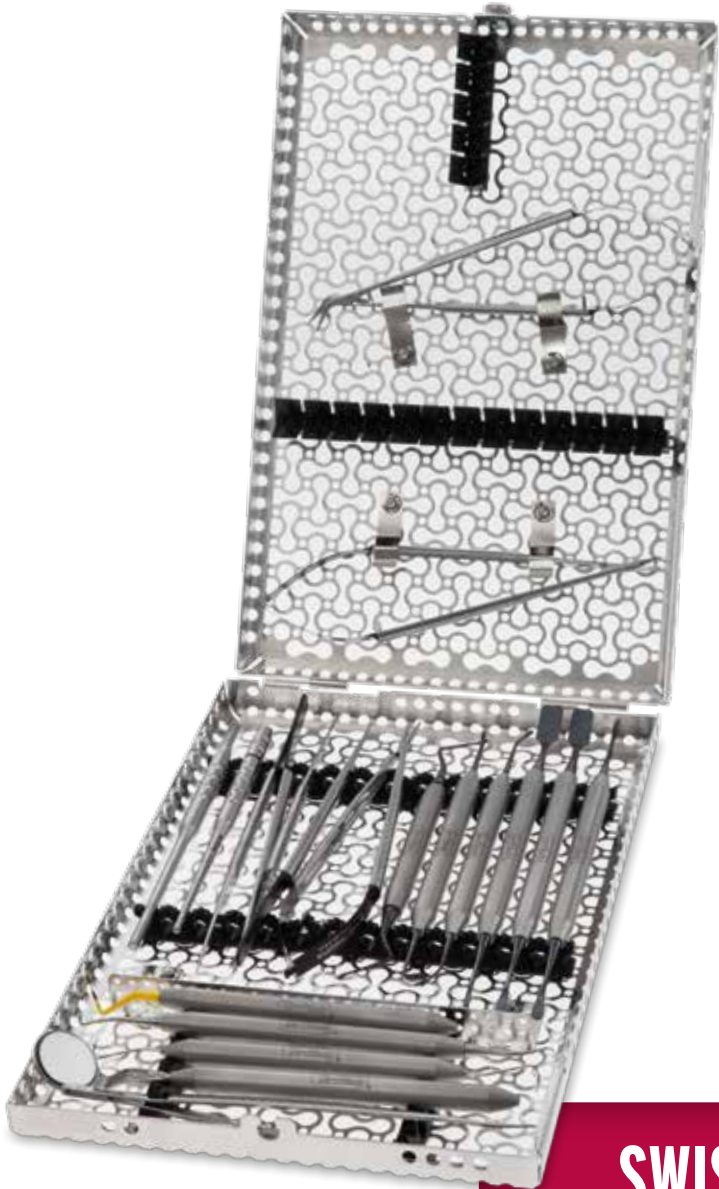

CONTAINS:

Code	Description
IMEDIN187S	IMS Infinity DIN Surgical cassette for 18 instruments, black
SPNHDPVN	Swiss Perio Microsurgical Needle Holder Stainless Steel
SPTPDSSPV	Swiss Perio Microsurgical Plier Straight Stainless Steel
SPTPDAPV	Swiss Perio Microsurgical Plier Straight Stainless Steel
SPSPV	Swiss Perio Microsurgical Scissor Stainless Steel
SPSPV45	Swiss Perio Microsurgical Scissors Stainless Steel, 45 degree
SPP24GSPX	Swiss Perio Periosteal Elevators Mod. P24G, Black Line
SPP24GSICX	Swiss Perio Periosteal Elevators Mod. P24GSIC, Black Line
SPPH2X	Swiss Perio Tissue Dissector Mod. PH2, Black Line
SPPSCHLEEX	Swiss Perio Papilla Elevator Mod. PPSCHLEE, Black Line
CL84X	#84 Lucas Double Ended Surgical Curette, Black Line
SPP24GSP2X	Swiss Perio Periosteal Elevators Mod. P24GSP, Black Line
SYG7/8CHE2	#YG7/8 Younger-Good, EverEdge 2.0, Smooth Handle Stainless Steel
PH6CH	Colorvue Probe Handle, Smooth Handle Stainless Steel
SHDPV	Mini Scalpel Blade / Mirror Handle Stainless Steel
10-130-05E	#5 European Style Round Scalpel Handle
MHE6CH	#6 Simple Steam Mirror Handle, Smooth Handle
DP17	#17 Dressing Pliers
PQ2NCH	#Q2N Color-Coded Nabers Probe, Smooth Handle
PCV12PT	#12 Colorvue Probe tips, 12 Replacements Tips
MIR5DSHDE	European Simple Steamn.5 Double-Sided HD Mirror, Single

SWISS PERIO KIT

1.403.036,31 FT (retail 1.454.719,21 FT)

LET'S BE CLEAR: ENHANCING ALIGNER TREATMENTS

Clear Collection instruments

678-800

The Tear Drop

678-801

The Vertical

678-802

The Horizontal

678-803

The Hole Punch

55.760,87 €
each



USE THE RIGHT CASSETTE

IMNORTH58

IMS Infinity Cassette Ortho,
small, thin, blue

72.544,94 €

In part 1 of a series, **Dr. S. Jay Bowman** explores instruments that help increase the utility of aligners and expand the scope of appropriate applications

As clear aligner treatments have evolved and been progressively refined in the past 15 years, more advanced applications and increasingly complex malocclusions have been addressed.¹⁻⁴

In addition, there has been more interest in tackling some of the specific limitations of moving teeth with plastic.⁵⁻⁷

Obviously, the progress of orthodontic treatment with traditional wires and braces has been advanced throughout the past century, but the techniques associated with a sequence of aligners are relatively recent phenomena.

Orthodontists with the most experience and enthusiasm in using aligners have worked diligently to improve the concept through better understanding of the limitations involved in more advanced treatment planning, especially when using adjuncts to enhance the associated biomechanics.^{4,8-13}

The Clear Collection of instruments from Hu-Friedy (Chicago, Illinois, at www.hu-friedy.eu) was developed with that aim in mind; namely, to help to increase the utility of aligners and expand the scope of appropriate applications.¹³ ►

HU-FRIEDY
PROMOTIONS



LET'S BE CLEAR: ENHANCING ALIGNER TREATMENTS

Clear Collection Instruments

“Innovation is taking two things that already exist and putting them together in a new way.”
Tom Freston

Adding intra- and intermaxillary forces

The Clear Collection instruments have streamlined and standardized some procedures that orthodontists had already been providing, while opening up more options for other methods to accentuate or refine treatment.

In the past, attempting to add intermaxillary elastics for correction of Class IIs, IIIs, or midlines was problematic.

Some were cutting notches in plastic trays using scissors or nail clippers¹⁰ or attempting to adhere buttons to the trays – all unwieldy endeavors.

The Tear Drop

There are occasions when orthodontic elastics are needed to assist with specific tooth movements or growth modification when using aligners. For instance, it may be beneficial to add elastic hooks or bonded buttons in different locations along the arch form on either the buccal or lingual. It would be beneficial to incorporate an elastic hook that cannot be dislodged from the aligner and that only requires a simple one-step squeeze of a pair of pliers to create. Most importantly, a hook that actually holds the elastic in place on the aligner.

The Tear Drop pliers (Figure 1) is an instrument created for the purpose of adding a notch or hook at the gingival margin of clear aligners. A standardized notch is easily cut in a single step, creating a teardrop-shaped “reservoir” to hold the elastic on the tray



| Figures 2A-2B: Teardrop-shaped hooks retain elastics when aligners are seated, making the addition of “rubber bands” easier for patients to manipulate. Notches are made at an angle to resist forces applied by the elastics



(Figure 2), thereby making it easier for the patients to seat their aligner and connect their elastics. In this manner, the patient is not fumbling with elastics, attempting to hook them in two locations since the elastic accompanies the aligner as it is seated over the teeth.

Teardrop notches should be cut at an angle to resist the line of force of elastics that are being employed (e.g., the notch is angled anteriorly at the mesial of the upper cuspid for Class II elastics; Figure 2). These hooks can be added anywhere along the clear aligner arch at the gingival margin on either the facial or lingual, whenever elastic forces need to be applied.



| Figures 1A-1D: The Tear Drop pliers was created to produce a notch in clear aligner plastic to facilitate the application of orthodontic elastics



The Hole Punch

The Hole Punch (Figure 3) is used to cut a half-moon shaped hole at the gingival margin of aligners. These half-circle cuts permit the addition of bonded buttons, bonded orthodontic tubes or brackets with associated hooks, or are simply used to relieve impingement of plastic on soft tissue. The Hole Punch can be used to produce cuts on either the facial or lingual of any tooth, anywhere along the aligners, whenever needed. Buttons bonded to teeth, rather than to the trays, do not cause dislodgement of the aligner.



| Figures 3A-3B: The Hole Punch pliers was created to produce a half-moon cutout to permit the addition of bonded buttons or brackets

During the progress of some patients' treatments (after a series of aligners has already been fabricated and delivered), an orthodontist may wish to add elastics. Bonded buttons can be added to hook up Class II or III intermaxillary elastics, bite-seating elastics, Class I intramaxillary elastics to miniscrews (Figure 4), or elastomeric chain connected to buttons to correct dental rotations. The combination of buttons on molars (plastic relieved using the Hole Punch) along with elastic hooks cut into the plastic at the cuspids (using the Tear Drop) is a common method of employing orthodontic elastics.

In other instances, aligners will occasionally impinge upon gingival tissues and cause gingival pain or inflammation. The Hole Punch can be used to clear the impingement of

plastic in each of a series of aligners. A common location for that type of irritation is at the incisive papilla, behind the maxillary central incisors (Figure 5). Nipping along the gingival margin of a tray with the Hole Punch is also an option to relieve marginal gingival impingement on either the buccal or lingual. ►



| Figures 4A-4B: A variety of methods to connect orthodontic elastics can be added to clear aligners using the Hole Punch and the Tear Drop instruments. 4A. Class I intramaxillary elastics connected from teardrop hook to a miniscrew (i.e., to support molar distalization) plus Class II intermaxillary elastics from teardrop notch to bonded button. 4B. Intermaxillary elastics hooked to bonded buttons to assist with seating of teeth into aligners and improve occlusion



| Figures 5A-5D: The Hole Punch pliers can be used to relieve plastic impingement of gingival tissues anywhere along the aligners (e.g., incisive papilla irritation)



LET'S BE CLEAR: ENHANCING ALIGNER TREATMENTS

Clear Collection Instruments

Bootstrap mechanics

If certain teeth are not "tracking" or are lagging¹³ behind (i.e., not fitting into the tray; Figure 6) Aligner Chewies[™] are employed (Chewies[™] Aligner Tray Seaters, Dentsply Raintree Essix, York, Pennsylvania). Patients are asked to hold the Chewie between the teeth in question and squeeze 10-15 seconds, release, and repeat for 5 minutes, 2-3 times per day^{8,13}. In addition, Chewies are routinely used for the first few days when patients switch to a new pair of aligners to help them seat more completely.



Figure 6: Aligner "lag" or lost tracking is most often characterized as an "air gap" between the incisal or occlusal of teeth and the plastic, indicating teeth are not following the prescribed tooth movement¹³

Another option is to add elastics to forcibly erupt the teeth into the aligner tray.^{13,14} There are several methods to generate so-called "bootstrap mechanics" (Figure 7):¹³

1. An orthodontic elastic is stretched over the plastic aligner to connect to a combination of bonded buttons on both lingual and buccal of the tooth.
2. A bonded button is placed on the lingual of the tooth (plastic cleared with the Hole Punch). The Tear Drop notches are cut into the tray at the gingival embrasure spaces in the buccal plastic on both the mesial and distal of the problematic tooth, and an elastic hooked from the buccal notches and stretched over the aligner tray to the button to produce an extrusive force.¹³

The Tear Drop and/or Hole Punch cuts are made in each tray in the series of aligners prior to their being delivered to the patient. This concept is also useful when the extrusion of a tooth is needed for restorative purposes.



Figures 7A-7C: Bootstrap mechanics¹³ to forcibly erupt a "lagging" tooth using orthodontic elastics (7A-7B). The Hole Punch is employed to clear aligner plastic to permit the addition of bonded buttons. The Tear Drop is used to cut notches in mesial and distal embrasures (8C-8F).

Other biomechanics that benefit from the application of elastic forces to aligner trays may include the correction of severe rotations, posterior or anterior intrusion mechanics for open bites,^{11,12}



Figures 7D-7H: The Tear Drop is used to cut notches in mesial and distal embrasures (7C-7F). Orthodontic elastics are stretched from the button to the teardrop notches or to another button on the opposite side of the tooth (7G-7H)

Clinical images are courtesy of Dr. Jay Bowman

and forces to control anchorage during molar distalization or en masse movements when employing miniscrews (Figure 4).¹³ Cutting notches and relieving the plastic to add buttons or brackets facilitate the application of these innovative mechanics.

It is important to remember that indiscriminate alteration of the integrity of clear aligners may reduce their structural strength or may impair the intended biomechanics programmed into an aligner.

The instruments in the Clear Collection help the orthodontist to better customize clear aligner treatments, enhance their desired biomechanics, and streamline the addition of adjunctive forces during the course of a series of aligners. For information on the use and applications of the Clear Collection, instructional videos are available on YouTube at

<https://www.youtube.com/watch?v=hrs2VfnlmLY>. ■



S. Jay Bowman, DMD, MSD, is a Diplomate of the American Board of Orthodontics and a member of the Edward H. Angle Society of Orthodontists.

He developed and teaches the Straightwire course at the University of Michigan, is an Adjunct Associate Professor at Saint Louis University, an Assistant Clinical Professor at Case Western Reserve University, and Visiting Clinical Lecturer at Seton Hill University.

He maintains a private specialty practice of orthodontics in Portage, Michigan.



REFERENCES

1. Clements KM, Bollen AM, Huang G, King G, Huijoel P, Ma T. Activation time and material stiffness of sequential removable orthodontic appliances. Part 2: Dental improvements. *Am J Orthod Dentofacial Orthop.* 2003;124(5):502-508.
2. Djeu G, Shelton C, Maganzini A. Outcome assessment of Invisalign and traditional orthodontic treatment compared with the American Board of Orthodontics objective grading system. *Am J Orthod Dentofacial Orthop.* 2005;128(3):292-298.
3. Phan X, Ling PH. Clinical limitations of Invisalign. *J Can Dent Assoc.* 2007;73(3):263-266.
4. Tuncay O. *The Invisalign System*. London: Quintessence Publishing; 2007.
5. Kravitz ND, Kusnoto B, BeGole E, Obrez A, Agran B. How well does Invisalign work? A prospective clinical study evaluating the efficacy of tooth movement with Invisalign. *Am J Orthod Dentofacial Orthop.* 2009;135(1):27-35.
6. Krieger E, Seifert H, Marinello I, Jung BA, Wriedt S, Jacobs C, Wehrbein H. Invisalign® treatment in the anterior region: were the predicted tooth movements achieved? *J Orofac Orthop.* 2012;73(5):365-376.
7. Chisari JR, McGorray SP, Nair M, Wheeler TT. Variables affecting orthodontic tooth movement with clear aligners. *Am J Orthod Dentofacial Orthop.* 2014;145(4 suppl):S82-91.
8. Tuncay O. *Clinical Reports & Techniques*. 2005;6(2):1.
9. Daher S. Techniques for Class II correction with Invisalign and elastics. Align Technology, Inc. Web site. Published 2013. Accessed April 11, 2015]. <https://s3.amazonaws.com/learninvisalign/docs/06840000006HgmAAG.pdf>
10. Paquette D. Temporary Anchorage Devices in Combination with Aligners 2009. Invisalign Web site. http://www.aligntechnology.com/files/ATEArchive/pdf/ate_110609.pdf
11. Boyd R. How successful is Invisalign for treatment of anterior open bite and deep overbite? Presented at: American Association of Orthodontists Annual Session; May 5, 2013; Philadelphia, PA. <https://www.aaoinfo.org/system/files/media/documents/Boyd,%20Robert%20--%20Treatment%20of%20Deep%20and%20Open%20bite%20with%20Clear%20Aligners.pdf>
12. Dayan, W. Techniques for Posterior Intrusion with Invisalign to Correct Anterior Open Bites. Techniques White Paper 2010. <http://www.cityortho.ca/Portals/0/City%20Ortho-%20Open%20Bite%20Blog.pdf>.
13. Bowman SJ, Celenza F, Sparanga J, Papadopoulos MA, Ojima K, Lin J. Creative adjuncts for clear aligners, Part I: Class II treatment. *J Clin Orthod.* 2015;49(2):83-94.
14. Giancotti A, Ronchin M. Pre-restorative treatment with the Invisalign system. *J Clin Orthod.* 2006;40(11):679-682.





TECHNICAL CARE CENTER EXTENDING INSTRUMENT LIFE PERFORMANCE

Dedicated Craftsmanship

A full set of services to maximise your instrument life and extend its performance

Sharpening A sharp instrument ensures successful outcomes, less fatigue and increases patient comfort.

Repairing Managed by a team of renowned technical experts to deliver timely repairs and reconditioning.

Refurbishing We aim to get your device or instrument back in your hands fast, in as close to 'new' condition as possible.

Polishing Polishing is the collection of high technology driven processes aiming at finishing instruments surfaces to improve their aesthetic quality.

Laser Marking Want your instruments or IMS Cassettes to be labeled with your name or your dental studio brand? We can do that.

Consultancy Tailored consultancy dedicated to your needs, capable of suggesting the best way to be the most effective with your instruments.

TUTTLINGEN
TECHNICAL CARE CENTER EMEA



Would you like to discover some tips and tricks to achieve a perfect isolation?

Do you know which are the preparation tools and how to use them?

Preheated composite, flowable composite or dual-cure cement for overlay composite luting?

CHECK THIS OUT AND MUCH MORE ON WERESTORE.IT* CHANNEL ON

HU-FRIEDY
PROMOTIONS



os osteocom
THE DENTAL LEARNING PLATFORM

Adhesive Dentistry Procedures

by Paolone and Scolavino



POWERED BY **Hu-Friedy**



Hu-Friedy Highlights - All prices are VAT included. Valid from February 14th until April 30th, 2020

Hu-Friedy




NEXT GIGS

COURSES/WEBINAR

- 1 **EEP PERIO MASTER CLINIC**
6-7/03/2020 - *Dublin*
- 2 **EXPODENTAL**
12-14/03/2020 - *Madrid*
- 3 **DENTAL EXPO AMSTERDAM 2020**
19-21/03/2020 - *Amsterdam*

- **Adhesive Dentistry Procedures**
by **Paolone e Scolavino**
on demand webinar

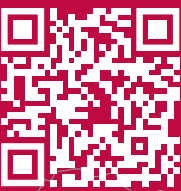


DOWNLOAD OUR CATALOGUE

For all of us at Hu-Friedy, this is much more than a book that contains the highest quality dental instruments and products. It's a manifestation of the dedication, innovative spirit and hard work of a company that has served and supported the dental community for over 110 years. catalog.hu-friedy.eu

ASK YOUR LOCAL REPRESENTATIVE OR DEALER FOR THE AVAILABILITY IN YOUR COUNTRY

As some of the products included in this catalogue might not be available in some countries, we invite you to verify the availability of the product with your local authorised dealer or to contact our customer care at the free phone number 00800 48 37 43 39.

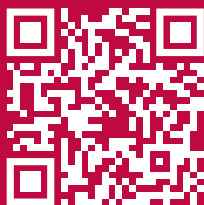

DOWNLOAD OUR CATALOGUE

For all of us at Hu-Friedy, this is much more than a book that contains the highest quality dental instruments and products. It's a manifestation of the dedication, innovative spirit and hard work of a company that has served and supported the dental community for over 110 years. catalog.hu-friedy.eu

FOLLOW US ON:



@HuFriedyEU



@hu_friedy_europe



@Hu-Friedy Mfg. Co., LLC.EU

For more information about
Hu-Friedy's Total Solutions visit
HU-FRIEDY.EU


PREVENTIVE



RESTORATIVE



ORTHODONTIC

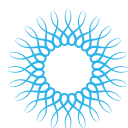


INSTRUMENT MANAGEMENT



ENDODONTIC



SURGICAL

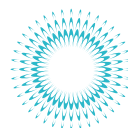

POWER



Manufacturer: Hu-Friedy Mfg. Co., LLC | 3232 N. Rockwell Street | Chicago, IL 60618 | USA
Customer Care Department: Hu-Friedy Mfg. Co., LLC.
Free Call +800 HUFRIEDY (00800 48 37 43 39) | Free Fax 00800 48 37 43 40
E-Mail: info@hu-friedy.eu | Website: Hu-Friedy.eu

©2020 Hu-Friedy Mfg. Co., LLC. All rights reserved. The contents of this Product Catalog & Reference Guide including, but not limited to, all text, photographs and graphics, are the sole and exclusive property of Hu-Friedy Mfg. Co., LLC and may not be copied, reproduced or distributed without prior written permission. Hu-Friedy Mfg. Co., LLC. reserves the right to change features of products, images and text at any time without notice. HFL-323HU/0220



How the best perform